

SDI

I
N
S
T
R
U
C
T
I
O
N
S

SDI poladay + polanight

E
N
G
L
I
S
H3%, 7.5% and 9.5% hydrogen peroxide +
10%, 16% and 22% carbamide peroxide
advanced tooth whitening systempoladay
polanight

ADVANCED TOOTH WHITENING SYSTEM

INSTRUCTIONS FOR USE

Pola Day and Pola Night are high viscosity, neutral pH take home tooth whitening gels. Both are pleasantly flavoured, crystal clear sustained release gels with a unique blend of soothers, conditioners and a water content high enough to further reduce sensitivity.

COMPOSITION:

Pola Day 3.0-9.5% wt Hydrogen peroxide
< 47% wt Additives
30% wt Glycerol
20% wt Water
0.1% wt Flavour

Pola Night 10.0-22.0% wt Carbamide peroxide
< 40% wt Additives
30% wt Glycerol
20% wt Water
0.1% wt Flavour

TREATMENT OPTIONS:

Pola Day 3%	7.5%	9.5%
a. 3 x 30 minutes/day	a. 2x 30 minutes/day	1 x 30 minutes/day
b. 1 x 90 minutes/day	b. 1 x 60 minutes/day	
Pola Night 10%	16%	22%
6-8 hours overnight	a. 2 x 40 minutes/day	1 x 40 minutes/day
	b. 1 x 90 minutes/day	

Please note, patients with sensitivity will require a day or two break between treatment. Shortening treatment time is also suggested.

INSTRUCTIONS:

Brush and floss teeth. Take a syringe out of the kit. Remove the cap and insert an application tip by twisting it securely onto the syringe.

- 1) Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
- 2) Make sure the tray is seated in the mouth.
- 3) Wipe away excess gel in mouth with finger, tissue or dry soft tooth brush.
- 4) After treatment, remove tray. Rinse tray and mouth with lukewarm water to avoid sensitivity.
- 5) Brush teeth.

PRECAUTIONS:

To be used under the supervision of a dentist.

Use at room temperature.

Do not use after expiry date.

Pola Day and Pola Night should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.

Pola Day and Pola Night may be irritating to skin and eyes. Wash with water if contact occurs.

Do not smoke during or immediately after the tooth whitening process. If you feel any sensitivity or unusual sensations, remove tray, rinse mouth and brush teeth. Discontinue the treatment and consult your dentist immediately.

When finished with the syringe replace tip with cap and secure tightly. Discard or in the case of a half used syringe, refrigerate. If tip is to be reused, wash it out with warm water.

The tray should be removed, mouth rinsed and teeth brushed before eating or drinking.




Foods and drinks containing strong colors should be avoided for at least 48 hrs or consumed in moderation during treatment, for example, coffee, soft drink or colored candies.

Keep out of reach of children and pets.

Do not ever inject Pola Day or Pola Night into the body.

Not to be used by pregnant or lactating women or children under the age of 14 years.

Keep unused syringes refrigerated.

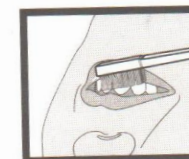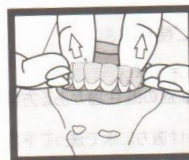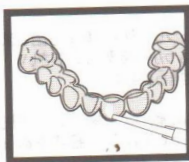

Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
USA & Canada 1 800 228 5166
Brazil 0800 770 1735
Germany 0800 100 5759
France 00800 022 55 734
United Kingdom 00800 022 55 734
Spain 00800 022 55 734
Ireland 01 886 9570
Italy 800 780625
Southern Dental Industries Limited
European Union Representative:
Block 8, St Johns Court
Swords Road, Santry
Dublin 9, Ireland
www.sdi.com.au

PAK41003 C